MEDICAL DEVICE MD CE
Designed and manufactured under medical supervision

MendivilTM
Fine Therapeutic Footwear

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EN

Mendivil was founded as a family business and its aim from the start was to become our customers' chosen brand in therapeutic footwear by providing innovative products of the finest quality and the best service. Our approach is based on ever greater respect and commitment toward our customers and suppliers, creating a healthy and sustainable environment thanks to the dedication of all our staff.

Mendivil

MDR 2017/745

Responsible cumplimiento normativa / Responsible for regulatory compliance:

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Current issue date: 5 May 2023
Expiry date: 4 May 2026
Certificate identity number: 10523836

Original approval(s):
ISO 9001 - 5 May 2023

Certificate of Approval

This is to certify that the Management System of:

Biomedical Shoes, S.L.

Poligono Industrial Rubial 13, 03400 Villena, Alicante, Spain

has been approved by LRQA to the following standards:

ISO 9001:2015

Approval number(s): ISO 9001 – 00040440

The scope of this approval is applicable to:

Design and manufacturing of therapeutic footwear.

Paul Graaf

Area Operations Manager, Europe

Issued by: LRQA Limited

LRQA Group Limited, its affiliates and subsidiaries and their respective officers, employees or agents are, individually and collectively, referred to in this clause as 'LRQA'. LRQA assumes no responsibility and shall not be liable to any person for any loss, damage or expense caused by reliance on the information or advice in this document or howsoever provided, unless that person has signed a contract with the relevant LRQA entity for the provision of this information or advice and in that case any responsibility or liability is exclusively on the terms and conditions set out in that contract.
Issued by: LRQA Limited, 1 Trinity Park, Bickenhill Lane, Birmingham B37 7ES, United Kingdom

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EN

The **CE marking** is the declaration of the medical device manufacturer that a product meets the essential requirements of European Directives and it is a **legal requirement** to market a medical device in the European Union. It represents the **sign of quality and safety** that obtain those products that meet the requirements to be used in patients with **all the guarantees**.

Mendivil has trained its staff and has adapted documentation, processes and facilities to the new Medical Device Regulation **MDR 2017/745** and has been certified to the standard **ISO 9001:2015** applicable to Design and Manufacturing of therapeutic footwear.

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FOOT TYPE

"AVERAGE" INSTEP,
NORMAL RANGE OF
MOTION (ROM)

Very suitable. 3 eyelets minimum. 4-6 eyelets creates a lower opening allowing easier entry and more adjustment.

Needs the facings to fit correctly over instep but good choice especially for men.

HIGH INSTEP AND
PES CAVUS

2-3 eyelets facings generally do not sit well on high-instep foot. 5-6 eyelets allow more ease of entry and adjustment. Look for extra depth lasts in "stock" (ready-made) ranges.

Poor choice for these foot types as facings cannot lay properly so foot appears to be "bursting" out of the shoe.

LOW INSTEP AND PES
PLANUS

3-5 eyelets styles are good but need to be fitted properly otherwise heel slip will occur. If facings butt together, shoe is too deep in instep circumference. May be solved with an extra in-sock or looking at alternative last proportions.

Can be successful for low-instep shape as long as facings do not overlap. With careful fitting, this style can help to hold foot firmly back into counter. Search for different last proportions until you find the right one.

HALLUX ABDUCTO
VALGUS (HAV)
AND LESSER TOE
DEFORMITIES

Extra width required for ease of entry. Lower opening facings with 4-6 eyelets are easier to get on. A very adaptable style - can have extra depth added for clawed toes or to accommodate the enlarged HAV joint.

Not suitable due to restricted entry.

HYPER-MOBILE FOOT

Great style that works well with control orthotics to give good support and stability.

Firm close fit of Oxford can be very successful here but search for best last proportions. Little room for orthotics.

LIMITED ROM OR FIXED
ANKLE POSITION

Gibson lace style is very suitable but needs to fit firmly to hold onto foot. Addition of rocker sole may be helpful but assess gait carefully first. Plenty of choice in stock ranges to find a good fitting last. Must have heel to ball flexion accurately fitted especially if adding a rocker sole.

Oxford style OK in general but entry might be difficult with fixed ankle position. If made with leather soles, can be very help as less flexibility and reduced ground grip aids limited gait movement.

OEDEMA

4-6 eyelets styles are best or twin Velcro straps. Add padded collar for extra comfort if swelling collects around the ankle. Extra depth may be needed and readily available in stock ranges. Keep heel height as low as possible for stability. May need extended medial heel for instep support.

Oxford is a poor choice for oedema due to limited opening and fixed base of facings. Style does not allow for swelling to increase over a few hours.

SEVERE VALGUS
DEFORMITY;
CHARCOT-MARIE-TOOTH

Choose a 3-6 eyelets style which may need to be modified or bespoke version to fit the deformity properly. Medial facings often need to be specially cut to accommodate the foot shape. Style can also be made to accommodate cradles or orthotics. Medial heel flare also very useful for control.

Oxford style is unsuitable, even if bespoke, due to limit of style over instep: does not work well when modified for a valgus foot shape.

DIABETIC

Gibson is an ideal style that can be modified to protect the diabetic foot with whole cut vamp and linings (seam free), wall toe puff (mudguard), very soft uppers. However, if the footwear is too soft there is a risk of the foot distorting the shoe and causing pressure points. By keeping a firm heel counter for control and when very carefully fitted, the diabetic foot can wear a firm shoe that does not cause any damage.

Oxford style is less suitable due to stitching and overlap at base of facings. A very difficult style to make seam free.

RHEUMATOID ARTHRITIS

Very suitable style. 3 eyelets minimum. 4-6 eyelets creates a lower opening allowing easier entry and more adjustment. Extra depth required for a reasonable depth of cushioning under the foot to toe end and keep lightweight and flexible by using a thinner sole (3mm) unit if necessary.

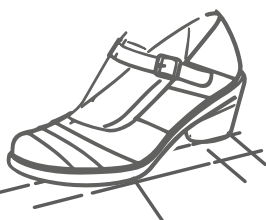
This style does not lend itself to be extra deep, and is also less flexible, with a leather sole, so is less likely to be a suitable choice.

AMPUTEES: SINGLE DIGIT
OR ALL TOES BACK TO
MIDTARSAL JOINT

Gibson style will work if the facings are cut to grip the foot as high as possible on the dorsum, as close to the front of ankle as is comfortable. Very firm fastenings required to keep shoe from slipping is vital. Partial foot filler can be placed inside shoe to compensate for short lever arm and prevent collapse of the upper and unsightly creases.

Oxford style can be very successful as it fits so well to the foot and grips on. Very careful fitting checks should be done to reduce risk of pressure.

03

MARY JANE

Position of strap is critical for comfort. A good style that many people can wear.

Not a good choice as strap can cause extreme pressure on dorsum of foot. This may damage veins or cause swelling to occur.

Depending on strap width and position, this style can work. A twin strap style is best. Check depth of vamp and instep as gaping may occur if too deep.

This style can work if made to correct depth and has a wide strap. Look for variety of last choice as many "stock makers" have this style in their selection.

Not suitable due to lack of instep control.

The Mary Jane is unsuitable for this foot type as it lacks support and is not adaptable to the change in foot position.

Not recommended. The swelling can bulge out through "Key hole" opening. Strap can cause major problems cutting into swelling. Wear with elastic stockings.

Mary Jane is unsuitable for this foot type as lack of support in instep and problems with gaping spoil the overall appearance.

Mary Jane can be used for diabetic feet provided the firm heel counter remains, and a twin strap would be best choice.

This is a popular style as it is lightweight and smart. Extra depth can be added to toe and vamp and, provided the strap holds firmly, it will be comfortable.

Mary Jane is unsuitable as it lacks body and good grip over the instep.

04

BOOTS

May limit ROM. The choice of fabric and soling affects the flexibility of boots.

Style of boots matters here as high-instep feet need an easy entry. Styles that have a side zip or front low lace are worth trying.

Most boot styles work well on low-arched feet. Plenty of choice available. Check foot is not sliding forward to end of toe puff.

Choice limited by boot styles that will go on easily. Low facing position. Velcro straps or laces best. Watch weight and flexibility as these may limit ROM too much.

A lace-up boot style is very helpful, gives good control and plenty of space for orthotics.

Boots can be helpful and supportive but entry can be difficult for limited ROM unless style is open to toe for ease of entry. Be aware of excess weight or bulk, e.g. padded collar and tongue, as these can compromise the little movement still available.

Boots with easily adjustable fastenings can be comfortable. However, in severe cases, boots are too cumbersome and bulky and, if oedema is variable, very difficult to fit around the ankles.

Boots are often the first choice and require a low opening for ease of entry. Very adaptable via modification or bespoke for most foot shapes. Good casts often needed and accurate measurements essential.

Boots are frequently prescribed for diabetic feet, as they are so adaptable, accommodative and protective. Watch weight and flexion of sole. Rockers often used as ROM is reduced in this style.

Boots can be heavy so the rheumatoid foot is less likely to be very comfortable in them. If ankle support is required, keep top height as low as possible and reduce weight by using lightweight outer soling.

Boots can be adapted for amputees. Allows great grip above the ankle and extra height helps to accommodate cradle or filler. Often necessary to add a rocker sole and extended shank to improve toe off.

05

TRAINERS

Very suitable but heel to ball fit needs to be correct. Depth at counter can cause pressure on Achilles tendon. Need to be properly fitted.

With careful selection, a trainer with a padded tongue should fit well.

Trainers with "built in" arch support may be quite uncomfortable for a low-arched foot. Check depth as too much space allows the foot to slide forward. Laces rather than Velcro are best for this foot type.

Not ideal as layers of stitching and seams often rub the joint or dorsum of toes. If made to order then depth can be adjusted over toes, and minimal "design features" to simplify construction helps.

Trainers are very suitable and work well with orthotics.

Trainers are great style especially if supplied with rocker soles and are properly fitted from heel to ball.

Trainers with padded tongue and collar enable this style to be very comfortable for oedema. Low opening laces are advantage for ease of adjustment.

Trainers can be adapted to suit if chosen with minimal design features. Can accommodate moulded cradle or orthotics. Very popular as foot shape can be disguised quite successfully.

Trainers are less suitable due to the number of pieces forming the upper causing extra stitching and seams. If very well designed and fitted with care this style can be used to satisfy patient compliance.

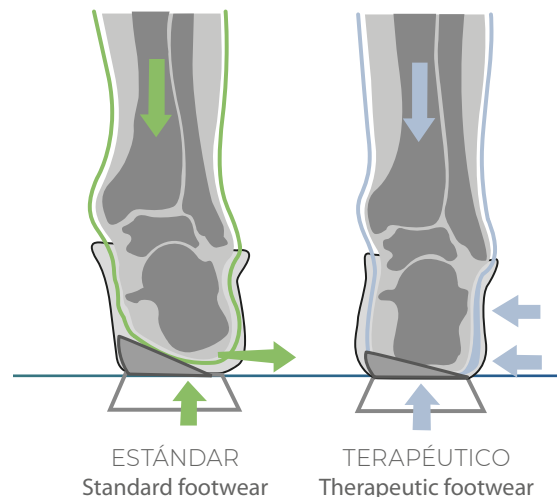
Trainers are less suitable due to the number of pieces forming the upper causing extra stitching and seams. If very well designed and fitted with care this style can be used to satisfy patient compliance.

Trainers can be used provided the lining is smooth inside to reduce any rubbing from extra seams on vamp. The ankle-high top style is more suitable combined with a padded collar and tongue.

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EN

INTERIOR SPACE:

The interior space of the footwear should perfectly accommodate the volume of the foot as well as the insole. This volume does not correspond to the dimensions of both, but there is an interaction from the weight of the patient and the pressure exerted by the insole on the foot.

COUNTER AND THE BACK:

The counter is longer and more rigid than in conventional footwear to stabilise the heel-bone and promote a normal walking pattern. The consistency of the counter should remain useful throughout the useful life of the footwear.

SOLE UNIT:

The sole of the footwear for insoles presents greater resistance against abrasion than in conventional footwear to assure that it will keep its shape after repeated use. Also, the heel is more rigid than in conventional footwear to prevent it losing its shape.

A worn-out or excessively soft sole can produce an incorrect alignment between the foot and the plantar orthosis which counteracts its effect, provoking a reduction in the effectiveness of the treatment.

COMFORT:

To improve the comfort of the footwear for insoles, we should:

- Control the height and design of the counter to avoid rubbing.
- Incorporate a padded tongue, padded collar, etc...
- Watch internal wear of the shoe to avoid sewing and reinforcement.

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Suggested prescription

Footwear + Orthoses

FOOT CONDITION	FEATURES OF ORTHOTIC THERAPY	SPECIAL FOOTWEAR FEATURES
HALLUX VALGUS SYNDROME	Orthotic with rearfoot control and valgus support and, if necessary, cushioning under metatarsal heads.	Footwear with additional girth for broadened forefoot, with a rim toe puff and deep vamp over toes. Ensure adequate heel-to-ball length.
PLANTAR FASCIITIS	Orthotic with valgus support, deep heel cup with base excavated and replaced with cushioning material.	Effective heel counter and waist.
ACHILLES TENDONITIS	Heel raise, gradually reducing in height as condition improves.	Shoe with deep heel seat to accommodate raise. Possibly a padded collar.
INTERDIGITAL NEUROMA	Rear-foot post to control pronation and forefoot component to off-load metatarsals adjacent to neuroma.	Forepart of shoe wide enough to accommodate all metatarsal heads and digits without compression. Low heel height, ensure functional fastening to minimize compaction of forefoot into the front of the vamp.
PES CAVUS	Total contact insole with reinforced valgus area.	Reinforced heel counter, adequate girth in tarsal region and adequate depth over toes. Lateral flare if necessary and consider rocker soles. Facings with 4-6 eyelets or more allow easier entry for the high instep. Padded tongue to reduce pressure on dorsum.
PES PLANUS	Total contact orthosis.	Strong heel counter with extension along medial side of midfoot. Medial flare if necessary.
RHEUMATOID FOOT	Orthotic with rear-foot control, posteriorly placed metatarsal bar and valgus filler. Cushioning under metatarsal heads.	Determine gait pattern. Where gait is totally apulsive, maybe rocker soles, but in the majority of cases, use flexible soling with adequate cushioning properties. Minimal seams in the forepart and deep vamp with rim toe puff to minimize trauma to deformed toes are indicated. Padded collar.
DIABETIC FOOT AT RISK	Total contact insole to equalize plantar pressures, laminated with firm base and cushioning top layer.	Shoe with rim toe puff. Deep vamp. Soft upper material containing no seams. Alternatively, very soft, seam-free full leather linings with a firmer upper material allow more control and reduce torsion that can occur if shoe is too soft. In all cases, a firm sole but not rigid with a rocker if required rather than too flexible to complement the orthotics.
CHARCOT DEFORMITY	Total contact insole with firm valgus control, but cushioned foot contact.	Preferably boots with reinforced heel counters extended medially into waist area. Rocker soles.
CHARCOT ROCKER BOTTOM FOOT	Total contact insole.	Preferably boots with reinforced heel counters extended medially into waist area. Rocker soles.
PARTIAL FOOT AMPUTATION	Total contact insole with block for amputated area.	May need shoe which extends high up the foot, or alternatively a boot. Ensure that the tread line of the shoe or boots is in the optimal position for gait - in the case of forefoot amputation, this may be within the blocked area of the footwear (consider the original shoe size before surgery). Rocker sole.
POSTERIOR TIBIAL TENDON DYSFUNCTION	Orthotic with valgus support, and posting for any associated forefoot varus.	Firm heel counter extended medially. In extreme cases, consider medial flare.
POSTERIOR TIBIAL TENDON RUPTURE	AFO and orthotic to support medial segment of the foot.	Footwear with adequate girth to accommodate AFO and orthotics. 4-6 eyelets for easier access with the AFO in position.

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EN- Models made with natural stretch leather or stretch materials and no internal seams, which adapts to foot prominences (bunions, hammer toes...).

Tallas / Ancho
Sizes / Width
35 / 43 / 16 M

MOD. 85.430

STOCK 24 HORAS / 24 Hours Delivery
NEGRO / BLACK

Tallas / Ancho
Sizes / Width
35 / 41 / 16 M

MOD. 85.160

STOCK 24 HORAS / 24 Hours Delivery
NEGRO / BLACK

Tallas / Ancho
Sizes / Width
35 / 41 / 15 k

MOD. 86.180

STOCK 24 HORAS / 24 Hours Delivery
MARRÓN / BROWN

Mujer Ladies

EN- Models made with natural stretch leather or stretch materials and no internal seams, which adapts to foot prominences (bunions, hammer toes...).

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EN- Sandals and Mary Jane style models made of natural stretch leather or stretch materials which adapt to deformities of the foot (bunions, hammer toe, etc...) and maintain the typical qualities of natural leather (breathability, adaptability).

Mujer Ladies

EN- Sandals and Mary Jane style models made of natural stretch leather or stretch materials which adapt to deformities of the foot (bunions, hammer toe, etc...) and maintain the typical qualities of natural leather (breathability, adaptability).

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EN- Footwear made with natural high quality leather which adapts to typical sport motions and movements and assures the adequate and effective use of the insole thanks to the special stiffener that stabilizes the rear foot.

Tallas	Ancho
Sizes	Width
25 / 46	13 H

MOD. 39.547

ENTREGA 15 DÍAS / Delivery 15 days

BLANCO-ROSA
 WHITE-PINK

Tallas	Ancho
Sizes	Width
25 / 46	13 H

MOD. 39.554

ENTREGA 15 DÍAS / Delivery 15 days

BLANCO-FUCSIA
 WHITE-FUCHSIA

Tallas	Ancho
Sizes	Width
25 / 46	13 H

MOD. 39.555

ENTREGA 15 DÍAS / Delivery 15 days

BLANCO-ROSA
 WHITE-PINK

Tallas	Ancho
Sizes	Width
25 / 46	13 H

MOD. 39.556

ENTREGA 15 DÍAS / Delivery 15 days

BLANCO-FUCSIA
 WHITE-FUCHSIA

Deporte Sport

EN- Footwear made with natural high quality leather which adapts to typical sport motions and movements and assures the adequate and effective use of the insole thanks to the special stiffener that stabilizes the rear foot.

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EN- Footwear made on a straight-shaped last with space inside to accommodate insoles, special stiffener, high quality soles and natural leather uppers-linings.

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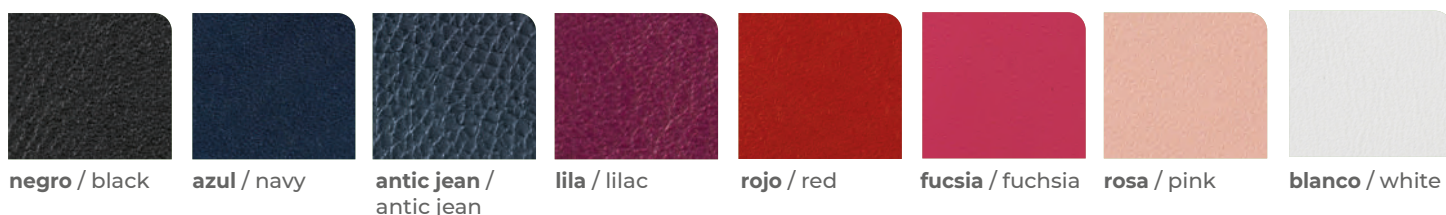
EN

Design your own shoes in 5 simple steps:

1. Select the model you like (see pages 47 to 55).
2. Choose the color you want in your shoes.
3. Select the suitable width fitting (see page 46).
4. Choose the suitable stiffener (ask for professional advice).
5. Select the sole unit that suits you and you like the most.

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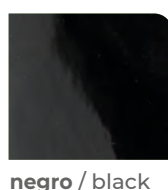
EN

It's important to note that when the foot weight bears, the foot posture changes and can have an effect on the shoe size required. Please consider this to ensure the desired room is added to the overall length (generally 1 cm).

Heel-to-ball length and flex angle: the joint at the base of the hallux (ball joint) must fit to the widest part of the shoe. This will ensure that the foot and shoe bend together.

CONTRAFUERTES STIFFENERS

ESTÁNDAR

STANDARD

ESPECIAL AFO

SPECIAL FOR AFO

ESTABILIZADOR EXTRAFUERTE

EXTRA STRENGTH STABILITY

OPCIONES DE PISO SOLE UNIT OPTIONS

PU

(Piso rígido)
Tallas 16-45

negro / black

beige / beige

gris / grey

rosa / pink

blanco / white

PU

(Rigid sole unit)
Available in sizes 16-45

GOMA MONTAÑA

(Piso semiflexible)
Tallas 18-45

negro / black

beige / beige

RUBBER MONTANA

(Semiflexible sole unit)
Available in sizes 18-45

GOMA OMEGA

(Piso flexible)
Tallas 16-42

negro / black

RUBBER OMEGA

(Flexible sole unit)
Available in sizes 16-42

GOMA KICK

(Piso flexible)
Tallas 20-45

negro / black

beige claro / light beige

RUBBER KICK

(Flexible sole unit)
Available in sizes 20-45

DEPORTIVO

(Piso semiflexible)
Tallas 16-45

negro / black

beige / beige

gris / grey

blanco / white

SPORTS

(Semiflexible sole unit)
Available in sizes 16-45

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EN- The styles below feature an additional zip fastening for ease of access and donning.

MOD. 714

Opción anchos 3 anchos de ortesis.
Last Options 3 orthoses widths.

Tallas / Sizes 20-45

Cierre Cordones o tipo Velcro® de ida y vuelta (con cremallera).
Fastening Lace or Reverse Pull Hook and Loop (with medial zip).

Rubber Montana
Goma Montaña
(semiflexible)

Black (with yellow stitch effect)

Rubber Kick
Goma Kick
(flexible)

Black (with yellow stitch effect)

MOD. 726

Opción anchos 3 anchos de ortesis.
Last Options 3 orthoses widths.

Tallas / Sizes 20-45

Cierre Cordones o tipo Velcro® de ida y vuelta (con cremallera).
Fastening Lace or Reverse Pull Hook and Loop (with medial zip).

Sports
Deportivo
(semiflexible)

Black

Grey

White

MOD. 735

DEPORTIVO
(Piso semiflexible)
Tallas 20-45

negro / black

beige / beige

gris / grey

blanco / white

SPORTS
(Semiflexible sole unit)
Available in sizes 20-45

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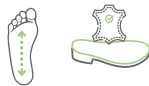
EN- Special boots for callipers with a low opening upper easy to fit and a rigid shank to stabilize the natural leather sole unit.

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EN

- 01 Position of the patient: lying down or seated with the leg extended and the foot at a 90° angle.
- 02 Place an elastic strip on the dorsal area of the foot will allow the later cutting of the plaster. (FIG. 1)
- 03 Apply pop bandage from the tips of the toes up to where the malleoli are covered. (FIG. 2)
- 04 Keep the foot in a 90° position and move the toes up gently until the plaster has set. (FIG. 3)
- 05 Mark a line from top to bottom in the same direction as the elastic strip from the ankle to the toes. (FIG. 4)
- 06 Mark various lines perpendicular to the first one. (FIG. 4)
- 07 Cut the mould along the longitudinal line, open it and take out the plaster boot. (FIG. 4)
- 08 Close the mould with more pop bandage positive it so that the perpendicular lines coincide.
- 09 Send the cast together with the measurements of the foot and a description of the footwear along with any other relevant information.

In case of equinus foot, with the forefoot in contact with the ground and the heel in full contact with the elevation blocks, draw a vertical line from the lateral malleolus proximally along the leg, and then a line at the base of this vertical which marks the true horizontal position. Another vertical line should be drawn at the back of the cast to ensure that the frontal plane position is accurate. (FIG. 5)

It is also useful to provide a patient's feet photograph.

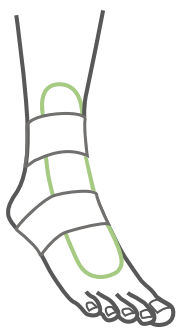
FIG. 1

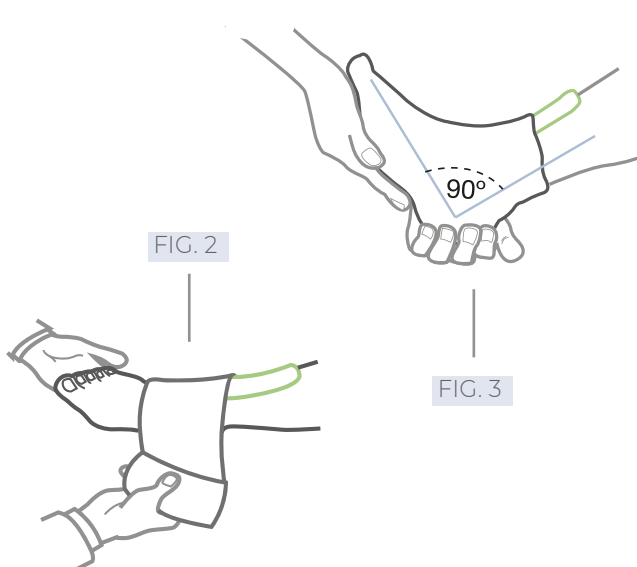
FIG. 2

FIG. 3

FIG. 4

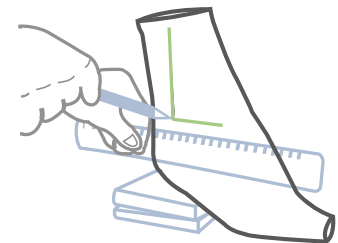
FIG. 5

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EN

MAKES POSSIBLE THE MANUFACTURE OF BESPOKE FOOTWEAR USING THE FOLLOWING EQUIPMENT:

01 - 3D FOOT SCANNER.

Using photographic and laser optic technology, the geometry of the foot can be taken in seconds.

02 - PROGRAM FOR LAST.

This program allows us to make a digital analysis of the measures and shape of the foot, as well as a comparative visual study with different lasts.

03 - PROGRAM FOR THE DESIGN OF THE MODEL.

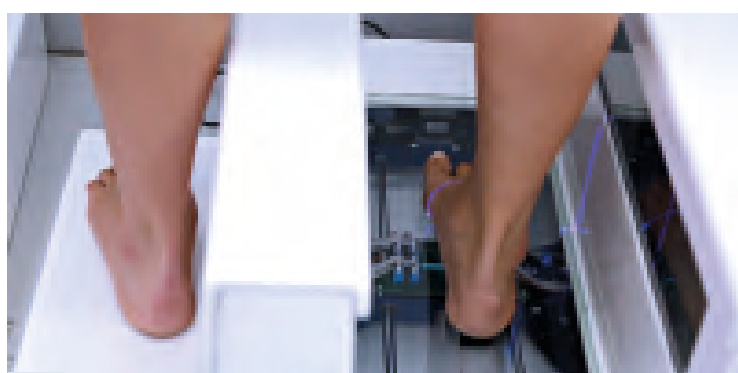
Allows the adaptation of an adjusted model on a standard last to a personalised last based on the patient. In this way we are able to make the patterns for the cutting process.

04 - MAKING THE PATIENT'S LASTS.

Once the last has been designed on the computer we transfer the data to a carve machine, which by means of a polishing action, makes a last based on the patient with an extreme degree of accuracy.

05 - MAKING THE MADE TO MEASURE FOOTWEAR.


After the cutting process we proceed to the lasting on the patient's last, the attachment of the sole and the finishing of the shoe.

01

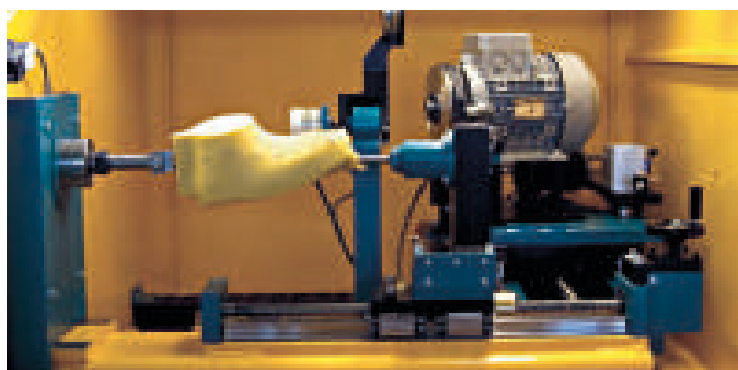02

03

04


Calzado a Medida Custom Made Footwear

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EN- Draft chart at the end of the catalogue.

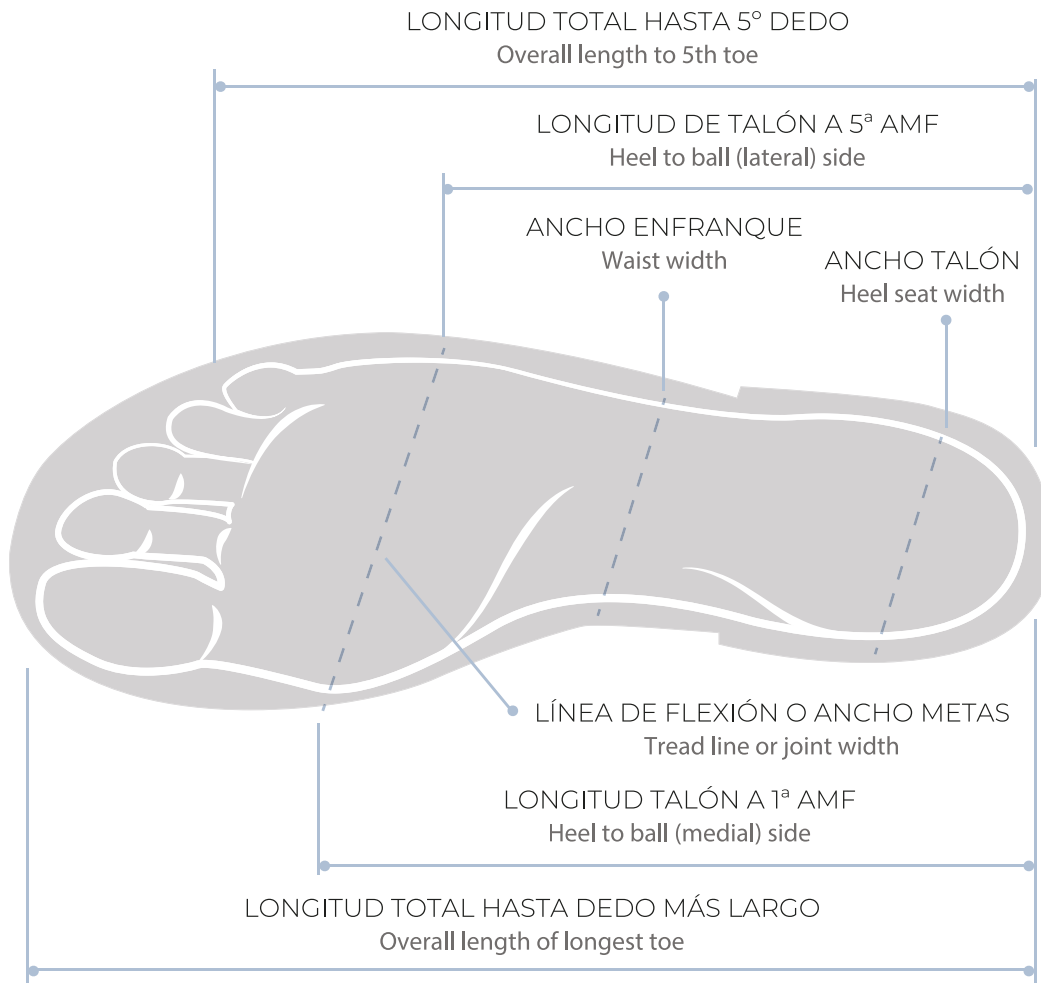Fitting problems & solutions in pages 74-75.

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EN

01 - WEDGES

Medial: when the medial aspect of the foot bears too much weight (genu valgum, toeing-in gait, excessive pronation).

Lateral: when the weight needs to be either supported or shifted from the lateral to the medial side of the shoe (genu varum, varus feet, tibial torsion, femoral torsion).

02 - ROCKER SOLE (BAR)

Take pressure off the metatarsal joints.

Conditions: to relieve pain, quicken or shorten the gait cycle. Assits dorsiflexion specially in fixed or limited ankle movement, relieve hallux rigidus. Use in conjunction with AFO's.

03 - THOMAS HEEL

The Thomas heel is manufactured with an anterior medial extension. Its function is to give support to the sustentaculum tali and the medial longitudinal arch. Use in conjunction with medial heel wedge in the treatment of valgus rear foot.

A reverse or lateral Thomas heel will support the cuboid area and will tend to rotate the foot externally (abduct).

04 - FLARED HEEL (FLOATED OUT)

A flare (or float) adds leverage to control the heel. It can be applied to either the medial or lateral side of the heel or extended along the whole sole of the footwear. It provides stability to the subtalar joint in unstable foot conditions (varus or valgus) and helps prevent recurrent sprains.

01

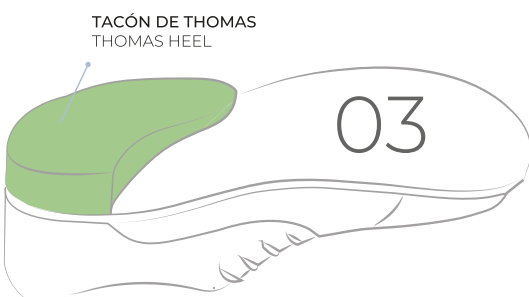
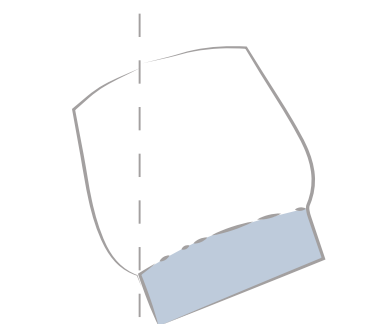
CUÑAS
WEDGES

02

PISO ESTÁNDAR
STANDARD SOLE UNIT


TACÓN DE THOMAS
THOMAS HEEL

03

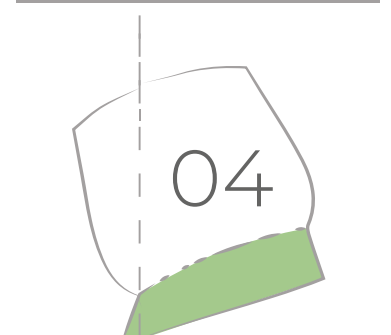PISO EN BALANCÍN
ROCKER SOLE (BAR)

04

TACÓN REDUNDANTE
FLARED HEEL

Calzado a Medida Custom Made Footwear

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Draft chart at the end of the catalogue.
Fitting problems & solutions in pages 74-75.

MOD. 4.101

CUERO / MID BROWN

MOD. 4.202

ARCE / ARCE

MOD. 4.203

RIOJA / BURGUNDY

MOD. 4.301

POLISANDRO / POLYSANDER

Calzado a Medida Custom Made Footwear

MOD. 4.302

MARRÓN / BROWN

MOD. 4.303

NEGRO / BLACK

MOD. 4.304

VAINILLA / VANILLA

MOD. 4.305

NEGRO / BLACK

JUANETERO

Bunion pocket

MOD. 4.306

NEGRO / BLACK

MOD. 4.307

NEGRO / BLACK

MOD. 4.308

NEGRO / BLACK

MOD. 4.309

MARRÓN / BROWN

Calzado a Medida Custom Made Footwear

MOD. 4.701

NEGRO / BLACK

MOD. 4.702

NEGRO / BLACK

MOD. 4.703

MARRÓN / BROWN

MOD. 4.704

NEGRO / BLACK

MOD. 4.705

NEGRO / BLACK

MOD. 4.706

NEGRO / BLACK

MOD. 4.707

NEGRO / BLACK

MOD. 4.708

NEGRO / BLACK

Calzado a Medida Custom Made Footwear

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
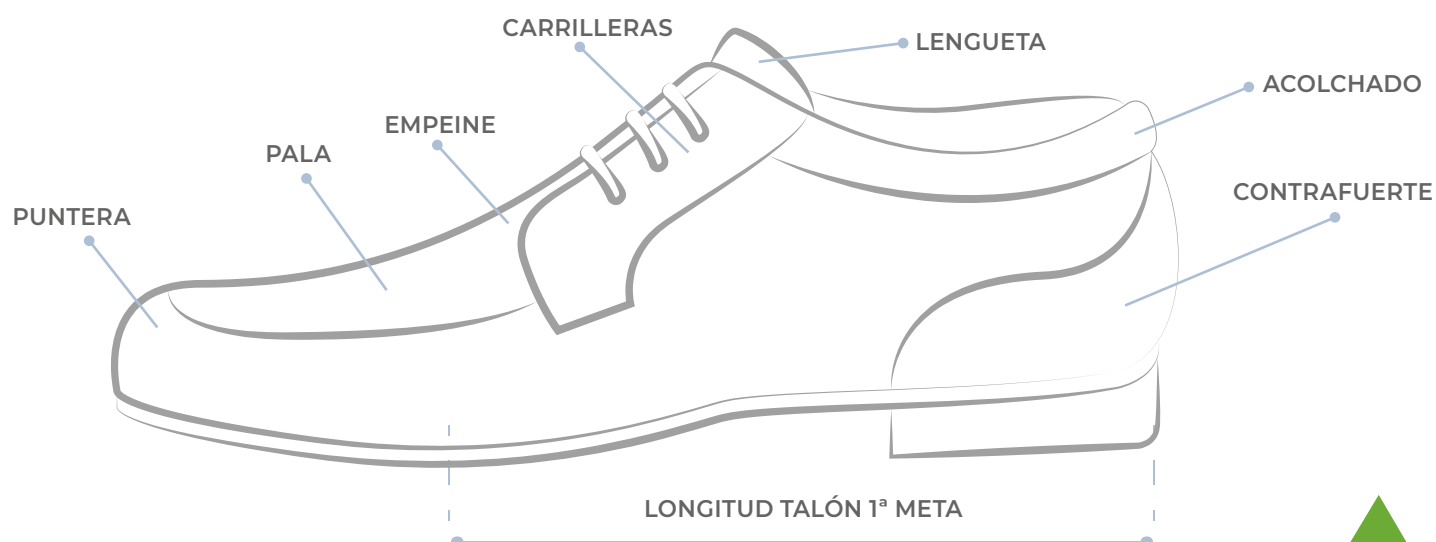
Fitting problems

and solutions

ÁREA OF SHOE	DIFFICULTY IDENTIFIED	VISUAL SIGNS	POSSIBLE CAUSES	SOLUTIONS
HEEL COUNTER	TOO HIGH	<p>Rubbing under either medial or lateral malleoli.</p>	<p>Height of malleoli not measured and stated on the draft. Unusually low malleolar heights.</p>	<p>Draw a line on the shoe counter with an effaceable leather pen, to show where it needs lowering. Give the measurement in mm, and request that the counter is lowered. This can be altered at fitting stage on bespoke or as an adaptation on stock items if there is no padded collar.</p>
			<p>Ankle fracture causing changes to bony prominences.</p>	<p>On bespoke items with a padded collar, request re-last of heel counter to lower height and state measurement required. Stock: check other makes to find one that is lower, rather than altering the shoe, especially if it has a padded collar. On a finished item, add 3 mm cork heel lift under inlay to lift heel higher in the counter to relieve any pressure. Ensure malleolar heights are recorded on future orders.</p>
		<p>Catching the Achilles tendon</p>	<p>Presence of 'heel bumps' or bursae over Achilles tendon.</p>	<p>Soften the counter with chemical softener, and use ring and ball stretcher to create a 'bump' in counter; encourage gradual 'wearing in' over a few days. Bespoke items can be ordered with a cavity in the counter stiffener to accommodate the bump.</p>
TOO LOW		<p>Does not grip the foot properly and foot comes out of the shoe too easily. Feels unsafe in wearing.</p>	<p>Counter depth sits too low on the foot.</p>	<p>On unfinished footwear, request counter depth to be increased and give measurements.</p>
			<p>Inlays may be too thick under heel, and lifting foot out. inside heel raise is too high for the counter.</p>	<p>Stock items: choose different maker. Reduce inside heel raise height, and add balance to external heel height, or change to external shoe raise only.</p>
			<p>Excessive oedema affects how the heel sits down in the counter.</p>	<p>Request modular or bespoke only. Straighten the back curve to accommodate oedema and increase width of heel seat.</p>
HELL-TO-BALL LENGTH	TOO LONG	<p>Footwear causes pain/discomfort; foot gets 'tired'. Creases too far back. Check for wear marks on sole behind flexion point.</p>	<p>Foot is flexing too far back in shoe, incorrect fitting.</p>	<p>If stock is being used, find shorter heel-to-ball last fitting. If bespoke, re-last on a last with a shorter heel-to-ball.</p>
	TOO SHORT	<p>Heavily creased vamp below facings. Toe puff may be creased as well. Toes often sore. 'Turned up' toe end. Heavy wear marks placed forward of flexion point</p>	<p>Foot is flexing too far forward in shoe, incorrect fitting.</p>	<p>If stock is being used, find longer heel-to-ball last fitting. If bespoke, re-last on longer heel-to-ball fitting. (This problem is rarely found with bespoke footwear if drafts and measurements have been taken correctly.)</p>
BALL JOINT	LIMITED RANGE OF MOTION (ROM)	<p>Twisted or angled creases that seem unusual. Wear marks heavy to lateral side.</p>	<p>Check first metatarso-phalangeal joint: ROM may be fixed or severely limited.</p>	<p>Add rocker sole and compensating heel to aid toe-off.</p>
INSTEP	TOO NARROW	<p>Facings pulled apart at wrong angle. Very tight; may not get shoe onto foot. Quarters bulge on medial side.</p>	<p>Poor quality drafts, no instep line or the measurements were not recorded correctly.</p>	<p>Redraw draft and ask for re-last.</p>
			<p>Excess pronation may cause poor foot position in shoe.</p>	<p>Corrective inlays may help. Uppers will need re-lasting to accommodate these inlays. (A quick way to check before casting is to use prefabricated neutral orthotics within the existing fitting and observe any improvements to foot position and/or facings).</p>
		<p>Incorrect style: does not suit foot shape or fasten properly.</p>	<p>Wrong style? Oxford facings can be very difficult to fit well, especially if instep is high. Instep bar or key-hole style can be shallow or narrow across instep.</p>	<p>Change style to Gibson or Derby instead. Reconsider style chosen, or redraft or cast to improve fitting.</p>
	TOO WIDE	<p>Facings feel loose; excess leather, particularly along medial side of quarter.</p>	<p>Fitting too wide. Entry probably too wide as well.</p>	<p>Change to narrower fitting, check foot on inlay for heel seat and instep fitting. Measure heel width: if it is narrow, it could also affect the instep.</p>
<p>Facings butt or overlap. Foot not retained adequately into counter.</p>		<p>Incorrect long heel measurement, or reduction in measurement due to oedema or change of dressings, etc.</p>	<p>Check measurements. Reduce instep circumference and/or entry giving measurement in mm. Remove inlays and check fitting again. Trim back inlays if faulty, until they fit correctly. Adding a valgus filler or a moulded inlay may save replacing the shoe.</p>	
		<p>Inlays too thin.</p>	<p>Increase inlays, but beware as this may compromise the fit elsewhere.</p>	
		<p>Insole board may be too wide at instep.</p>	<p>Reduce width of insole board; draw line on underside of shoe to show where it should be reduced. Give measurements.</p>	

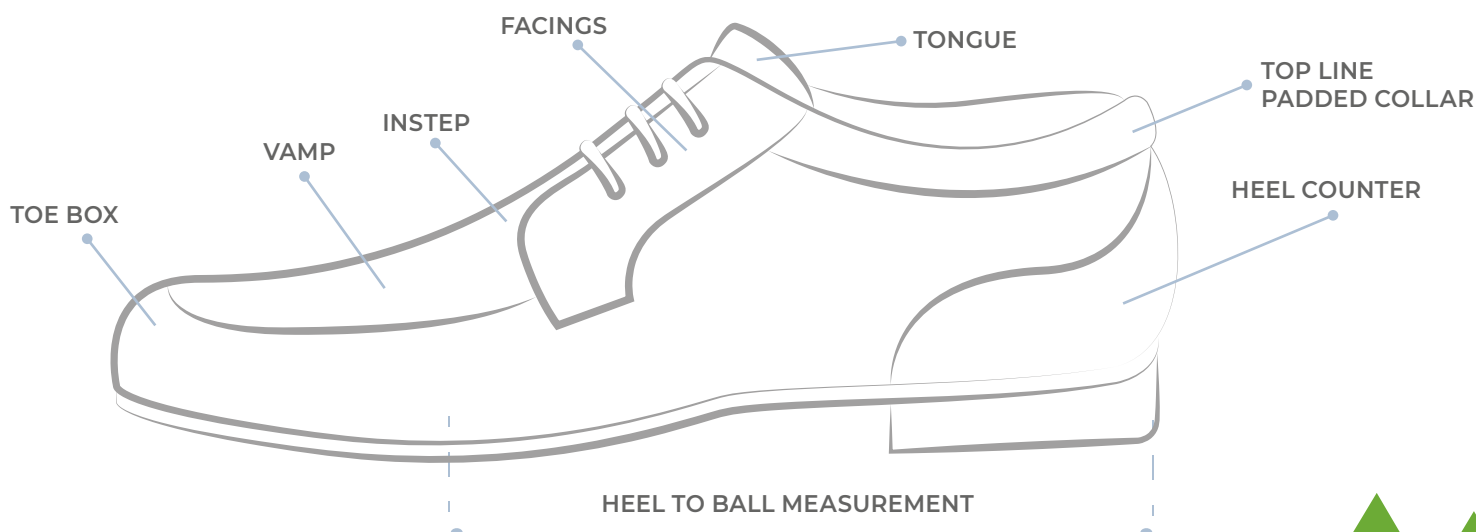ÁREA OF SHOE	DIFFICULTY IDENTIFIED	VISUAL SIGNS	POSSIBLE CAUSES	SOLUTIONS
VAMP AND TOE BOX	TOO WIDE	Feels big.	Change in measurements, or poor-quality draft supplied.	Check draft: remeasure if necessary.
		Excessive space around the toes.	Change in oedema or in hosiery or dressings.	If oedema is variable, then allowances are needed. Redraft and request 'add swelling corks' (removable 3 mm cork inlays that can be added or removed).
		Deep fold creases on flexion.	Insole board is too wide. Excess leather.	Feel position of the forefoot in vamp. Draw with effaceable leather pen where the excess is situated. Request reduction of insole board in mm. Check depth as this may also need to be reduced. This is possibly depth not width. Feel the position of the foot in the vamp: if width is correct under first to fifth joints, then reduce circumference.
	TOO DEEP	Feels 'baggy'. Excess leather across dorsum. Deep creases on flexion.	Change in measurement; reduction of oedema.	Reduce whole forefoot area by 'lasting down' and refit. If joint width is correct, reduce distal to the joint circumference.
		Base of facings bulge up from vamp.	Instep/entry is also too deep.	
	TOO NARROW	Feels tight.	Toe box shape of shoe not compatible with foot shape.	Compare foot shape with shoe; request change in toe box shape, if necessary, to rounded or square. Use inlay to check for width.
		Forefoot squeezed.	Swelling may have occurred.	Refit once swelling has reduced, or monitor swelling until stabilized.
		Stretch creases.	Insole board too narrow.	Check insole board against foot and request increase in width if too narrow. Mark shoe with effaceable leather pen where more width is needed.
		Toe puff may press into toes. Toes bulge into upper on lateral border.	Too narrow in toe box, or toe box wrong shape for foot.	Request change of toe shape and re-last the upper. (Note: increasing the width will also increase the circumference.)
	TOO SHALLOW	May not get shoe on.	Incorrect measurement.	Check draft, remeasure and request increase in depth giving measurement in mm.
		Very tight.	Change in measurements.	Request increase in depth, giving measurement.
		Toes rub on vamp leather.	No toe box depth measurement given on draft.	Solution as above.
One high toe may be pushing up on the upper.		Position of high toe not given on draft.	Mark tight area with effaceable leather pen, circle the area to be increased and request 'blocking up' of tight area, if rest of toe box depth is sufficient. Give measurement.	

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